



How to use

As an autogenous bone collector used for bone grafting during dental implant surgery, it harvests bone using a drill.

• Product Mix •

NLBC Ø 2, Ø 3, Ø 4, Ø 5 / Stoppers by each size / Bone pusher

• How to use(procedure for placement of 3 dental implants) •

- ① Connect an NLBC in the size an operator want to the engine from and then connect the stopper in order.
- ② Place the connected NLBC on the area that bone will be collected and conduct drilling. (recommended speed: 500RPM)
When the stoppers in are inserted completely, the insertion depth is 4mm.(identical in all sizes)
- ③ After drilling, separate the stopper and then push bone inside the stopper out using the bone pusher.

• Storage and maintenance after use •

- ① Clean the instrument with flowing water from body fluids or blood.
- ② Air dry away from direct heat.
- ③ Store in a dry place after sterilization.

• Directions for use •

- ① Be careful not to be injured as the blade of the drills is fine sharp.
- ② Make sure to sterilize it before use.
- ③ Make sure to be well-informed of the manual before use.

